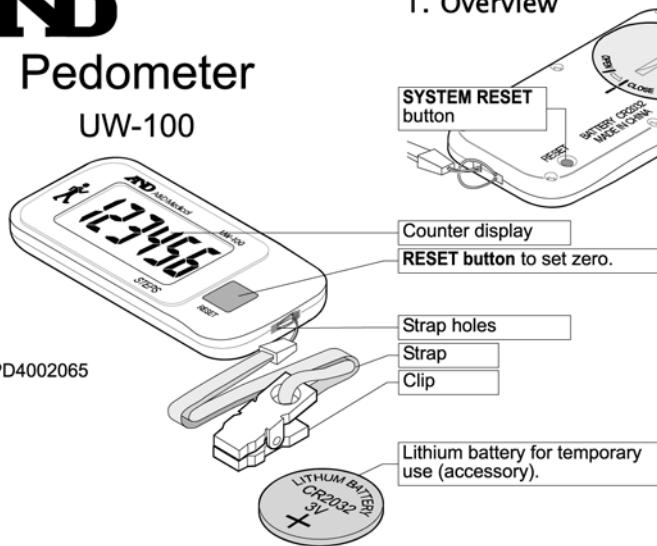
# Pedometer

## UW-100

1WMPD4002065

### 1. Overview

### 2. Features

- Displays the number of steps.
- Equipped with a 3-axis accelerometer to detect motion.
- When the pedometer is idle for one minute, the display turns off. If you start walking or press a button, the display turns on again.

### 3. Using the Pedometer

#### 3.1. Setting up the Pedometer

- Check the CR2032 battery**  
If you are using the pedometer for the first time or the  $L_o$  mark is blinking, refer to 3.3. & 4. .
- Attach the pedometer** ..... Refer to 3.2.  
Secure the pedometer to monitor body movement.
- Start measurement of walking (or moderate jogging)** ..... Refer to 4.  
  - To reset the display to zero, press the **RESET** button for a few seconds.
  - To ensure correct measurement, walk at a constant and continuous pace. Avoid excessive foot impact.
- Check the display**  
The display shows the number of steps.

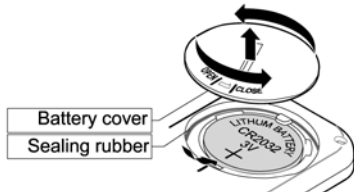
#### 3.2. Securing the Pedometer

- Attach the pedometer to detect motion.
- Secure the pedometer to prevent it from striking anything.
- Use the accessory strap and clip to prevent the pedometer from being dropped.
- Do not put the pedometer in the rear pocket of trousers.
- Avoid hanging the pedometer from the waist.
- To ensure correct measurement, avoid swinging the pedometer.

#### 3.3. Installing or Replacing the CR2032 Battery

- When using the pedometer for the first time, install the accessory battery.
  - When the  $L_o$  mark blinks or is displayed, replace the battery with a new one.
- Open the battery cover with a coin or screwdriver.
  - Replace the battery with a new one. Make sure the positive side (+) of the battery is facing up.
  - Return the battery cover and fasten it.
  - Proceed to the Step 2 under 3.1. .

### 4. Precautions

- Keep the pedometer out of reach of young children. If a young child swallows a part of the pedometer, consult a doctor immediately.
- When removing the battery, the stored data is deleted.
- Avoid touching the top (+) and bottom (-) of the battery at the same time to prolong the battery life.
- Dispose of a used battery in accordance with the local regulations for battery disposal.
- In the following cases, the pedometer may not measure correctly.
  - Excessively slow or very low-impact walking.
  - Vehicle movement : Automobile, train, bicycle.
  - Irregular motion : Exercise, swinging the pedometer.
  - Extreme impact : Jogging, jumping, rope skipping, running.
- The pedometer is not waterproof.
- Avoid extreme temperatures and humidity, direct sunshine, and dust.
- Do not drop or strike the pedometer.
- Do not open or repair the pedometer as electrical components may be damaged.

### 5. Symbols

Symbols	Function / Meaning	Recommended Action
	The low battery indicator is displayed when the battery power is low.	Replace with a new CR2032 lithium battery.
	The low battery indicator blinks when the battery is drained.	
	Direct current	_____
	EC directive medical device label	_____
	WEEE label	_____
	Manufacturer	_____
	EU-representative	_____

### 6. Troubleshooting

Problem	Possible Reason	Recommended Action
Nothing appears on the display, even when a key is pressed.	Drained battery. Battery incorrectly installed.	Replace the battery. Refer to 3.3. . Reinstall the battery. Refer to 3.3. .
Incorrect number of steps.	Incorrect detection.	Change the attaching style and walking style. Refer to 3.2. & 4. .
Nothing appears in the display at the start of walking.	Data is displayed after 10 consecutive steps to ensure correct measurement.	

**Note:** If you cannot fix the problem using the troubleshooting listed above, reset the pedometer using a pin or similar object to gently depress the **SYSTEM RESET** button. If you still cannot fix the problem, request service from your dealer.

### 7. Maintenance

- Clean with a soft dry cloth or a cloth moistened with water.
- Never use alcohol, benzine, thinner, or other harsh chemicals.
- Never wash with water or place in water.
- Remove the battery if the pedometer is not to be used for a long time.

### 8. Technical Data

Type	UW-100	
Display	Number of steps	0 to 999999
Measurement accuracy	±3%	
Power supply & Battery life	CR2032, approx. one year (at 10000 steps / day)	
Dimensions & Weight	75[W] x 34[H] x 11[D] mm, approx. 25 g (including battery)	
Operation condition	+10 °C to +40 °C, 30 %RH to 85 %RH	
Storage condition	-20 °C to +60 °C, 10 %RH to 95 %RH	

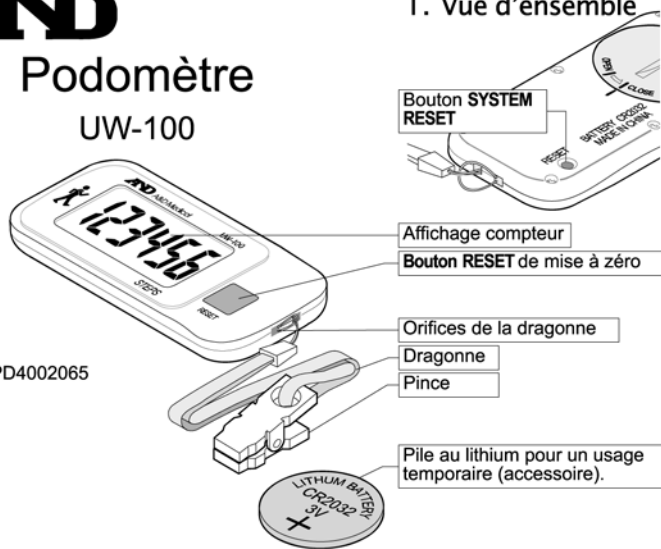
**Note:** Specifications are subject to change for improvement without prior notice.

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A&D Company Limited
**EC REP** A&D INSTRUMENTS LTD.
A&D Australasia Pty Ltd.